ARJOHUNTLEIGH

GETINGE GROUP

SARA STEDY

ENHANCED STANDING AID



...with people in mind







ENHANCING SUPPORT FOR MORE PATIENTS

Sara Stedy®, a new enhanced standing aid, builds on the success of Stedy, a proven mobility-promoting support aid that encourages more mobile patients and residents to stand up independently. With a number of new enhancements, Sara Stedy helps to minimise manual handling by carers in an even wider range of care situations. With an increased inner width, a new seat shape and a safe working load raised to 182 kg, Sara Stedy also meets the standing assistance needs of larger patients. Enhanced features including a pivoting divided seat and pedal-operated chassis legs ensure improved comfort and support, and make working routines easier for carers.



The innovative pivoting seat improves transfer efficiency and patient stability.



Pedal-operated chassis legs adjust easily for optimum access during transfers.



New total-lock castors provide complete stability when the aid is stationary.



Built-in seat handgrips

Handgrips integrated in the seat allow the carer to easily turn the support aid.



Provides extra stability or firm support when a patient cannot fully control leg movements.



Smaller castors and redesigned legs mean a lower chassis for better access.







A VERSATILE STAND AND TRANSFER AID

A better alternative to a wheelchair

The enhanced standing aid concept has become well established in a wide range of care situations in elderly care and healthcare facilities because it:

- offers a more suitable and dignified alternative to a wheelchair for more mobile residents
- promotes dignity and independence, as optimum participation in activities is possible
- enables good, reassuring eye contact between resident and carer, unlike a wheelchair
- provides transport to the toilet and washbasin, and can support other activities of daily living

Sara Stedy encourages patients and residents to fully participate in transfers, thus improving circulation, breathing, digestion and muscle tone. When at the toilet, the resident/patient is encouraged to stand, supported by Sara Stedy. The resident's/patient's independence and dignity is actively promoted. When required, the carer can assist with adjustment of clothing in an easy and safe way.

FOR QUICK, EASY AND SAFE TRANSFERS



Approach patient, open seat flaps, open chassis legs and place the patient's feet on the footboard.




Position Sara Stedy; apply castor brakes; raise patient with the help of the integrated handlebar; swivel seat flaps into position.




Release the brakes and proceed with the transfer

PRODUCT SPECIFICATIONS

Space Requirement

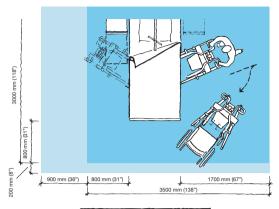

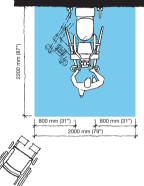

- Blue area shows the minimum working area required for the staff to be able to use the mechanical aids in an ergonomic way from one side.
- Light blue area shows required extension of working area to facilitate activities from either side to provide adequate access for the resident, mechanical aid and assisting carer

Mobility Gallery

Suitable for Albert, Barbara and Carl.



- A Albert, who is ambulatory and independent.
- B Barbara, who can support herself to some degree.
- C Carl, who sits in a wheelchair and has little capacity to support himself. Please contact ArjoHuntleigh for further information on the Mobility Gallery™.

ARJOHUNTLEIGH

www.ArjoHuntleigh.co.uk

ArjoHuntleigh UK Service and Technical Centre ArjoHuntleigh House,

St Catherine Street, Gloucester, GL1 2SL. United Kingdom Phone: +44 (0) 8456 114114

Service Fax: +44 (0) 1452 525207 Email: ukservice@ArjoHuntleigh.com

Email: dublin@ArjoHuntleigh.ie www.ArjoHuntleigh.ie

ArjoHuntleigh (Ireland) EA House, Damastown Industrial Park, Mulhuddart, Dublin 15, Republic of Ireland Phone: 01 809 8960 Fax: 01 809 8971

intensive care.

ArjoHuntleigh (Ireland)

19 Heron Road, Sydenham Business Park, Belfast, BT3 9LE, Northern Ireland Phone: 028 9050 2000 Fax: 028 9050 2001

Email: belfast@ArjoHuntleigh.com

29.4 kg (64.82 lbs)

182 kg (400 lbs)

905 mm (32 ¾")

1051 mm (41 ½")

729 mm (28 ¾")

100 mm (4")

www.ArjoHuntleigh.ie

Product Information

Max. safe working

Height of chassis

Internal width legs open

Lifter - Protection class IPX4

applicable product standards

® and ™ are trademarks of the ArjoHuntleigh group of companies

Compatible ArjoHuntleigh Slings and Flites (disposable slings).

The ArjoHuntleigh Mobility Gallery provides a general indication of the mobility categories that can benefit from using Sara Stedy. However,

support aid selection should always be based on an individual clinical

assessment by a doctor, registered nurse, certified clinician or similar,

GETINGE GROUP is a leading global provider of products and systems that contribute to quality enhancement and cost efficiency within healthcare and life sciences. We operate under the three brands of ArjoHuntleigh, GETINGE and MAQUET. ArjoHuntleigh focuses on patient mobility and wound management

solutions. GETINGE provides solutions for infection control within healthcare and contamination prevention within life sciences. MAQUET specializes in solutions, therapies and products for surgical interventions, interventional cardiology and

Low friction castors, the rear two with brakes

The product are in compliance with current legislation and

Only ArjoHuntleigh designed parts, which are designed specifically for the purpose, should be used on the equipment and products supplied by ArjoHuntleigh. As our policy is one of continuous development we reserve the right to modify designs and specifications without prior notice.

Weiaht

Total length

Total height

Approvals

© ArjoHuntleigh, 2011.

Accessories

Using Sara Stedy

GETINGE GROUP

ArjoHuntleigh UK - Head Office

Houghton Hall Business Park, Houghton Regis. Bedfordshire. LU5 5XF, United Kingdom Sales Phone: +44 (0) 1582 745700 Sales Fax: +44 (0) 1582 745745 Rental 24hr Helpline Lo-call Phone: +44 (0) 8457 342000

Rental Fax: +44 (0) 1582 745843 Email: sales.admin@ArjoHuntleigh.com